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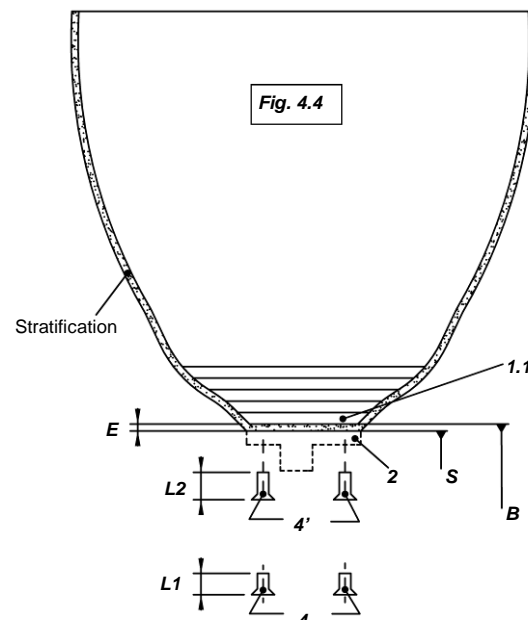
A) USE

This knee is designed for patients whose weight added to any load being carried does not exceed 100 kg.

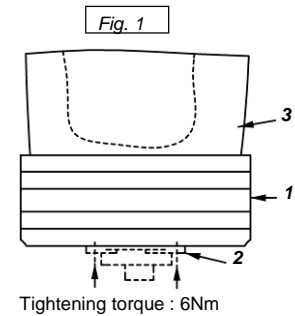
B) CONNECTIONS (Fig. 1)

1K06 or 1K10 (1) module must be connected :

- at its lower part to any European standard component (2) (4 holes for M6 countersunk head screw with 36 mm centre distance)
- at its upper part to a laminated or wooden socket (3).

C) PREPARATION (Fig. 1)

- Install 1K06 or 1K10 module (1) on the selected link piece (2).
- Align socket (3) statically.
- Glue socket (3) and module (1) with **PC232-5** adhesive resin (and **PC233-R** hardener) mixed with **XX022** talc (20 g of talc for 100 g of adhesive resin).


CAUTION!

- **The contact surface (glueing surface) between socket (3) and 1K06 or 1K10 module (1) should be at least 4/5 of the surface of the flat side of this module.**
- **Minimum drying time before handling the system : 30 min**
- **Minimum drying time before patient weight bearing : 1 H 30.**

D) ADJUSTMENTS (Fig. 2)

Use the 2 mm hex wrench delivered with the system.


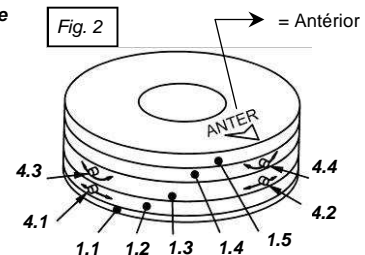
- **ANTERO-POSTERIOR TRANSLATION** (maximum range : $\pm 25\text{mm}$)
Loosen screw (4.1). Make the desired adjustment by sliding plate (1.2) onto plate (1.1). Tighten screw (4.1) to lock the adjustment.
- **MEDIO-LATERAL TRANSLATION** (maximum range : $\pm 25\text{mm}$)
Loosen screw (4.2). Make the desired adjustment by sliding plate (1.3) onto plate (1.2). Tighten screw (4.2) to lock the adjustment.
- **FLEXION-EXTENSION** (maximum range : $\pm 5^\circ$)
Loosen screw (4.3). Make the desired adjustment by sliding plate (1.4) onto plate (1.3). Tighten screw (4.3) to lock the adjustment.
- **ABDUCTION-ADDUCTION** (maximum range : $\pm 5^\circ$)
Loosen screw (4.4). Make the desired adjustment by sliding plate (1.5) onto plate (1.4). Tighten screw (4.4) to lock the adjustment.

CAUTION! For walking tests, it is essential to strengthen the assembly with reinforced adhesive tape or plastered bandages.

E) "STANDARD" FINISHING (Fig. 3.1, 3.2, 3.3, 3.4)

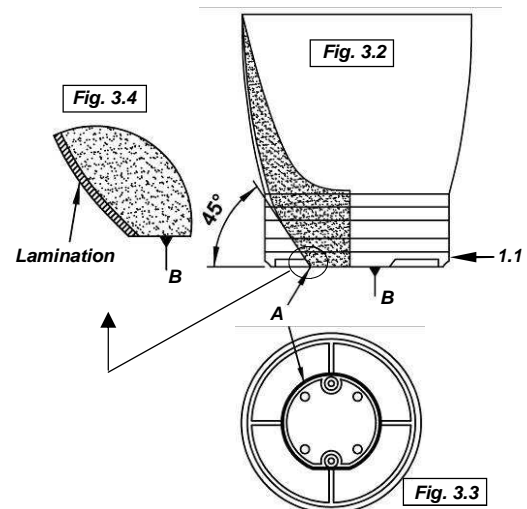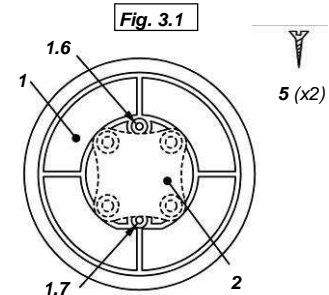- Remove link piece (2) and drill two \varnothing 2 mm holes at locations (1.6) and (1.7) of module (1) (fig. 3.1)
- Tighten in these holes the two screws (5) delivered with the system (fig. 3.1)
- Remove locking screws (4.1), (4.2), (4.3) and (4.4) (fig. 2)
- Cut off the excess material following the outer side of outline (A) and having an outward slope of about 45° on at least the height of plate (1.1) (fig. 3.2 and 3.3).
- Sand down and shape
- Mask the holes of lower surface (B) of the alignment system with adhesive tape for example (fig. 3.2, 3.3 and 3.4)
- Laminate the assembly with your standard procedure.
- When resin is hard enough, cut off excess material from the base of alignment system (1). Lamination must necessarily reach face (B) without covering it (fig. 3.4).



IMPORTANT! The type of finishing described in the above chapter E is the one that must be used for most of the socket shapes. For particular socket shapes, please read the following chapters.

F) PARTICULAR CASES

F.1) EXAMPLES

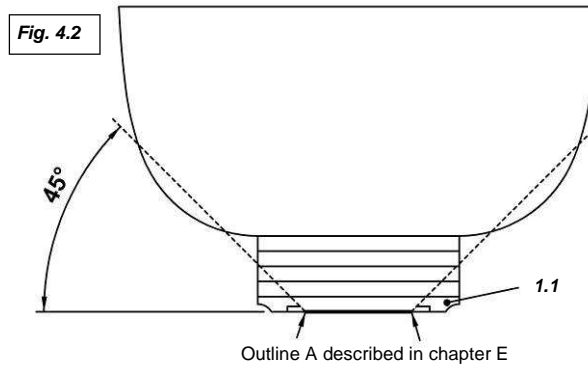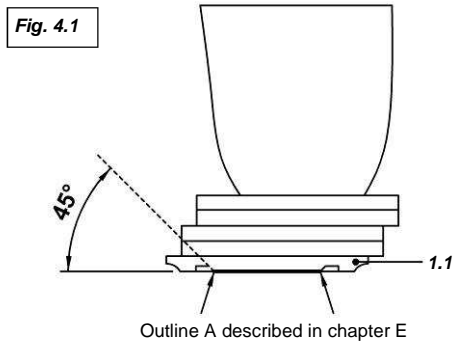In some cases (see the two typical cases of fig. 4.1 and 4.2), the finishing described in chapter E can not be performed because of the shape, and/or the size, and/or the socket alignment (it is not possible to cut off material excess according to outline **A** and/or to have a 45° cutting angle on a sufficient height).


F.2) "PARTICULAR" FINISHING

F.2.1) SHAPING :

a) *Cutting angle* : The cutting angle can be adapted to the shape wished for aesthetic matters. However a 45° angle is recommended, at least on the height of part **(1.1)**.

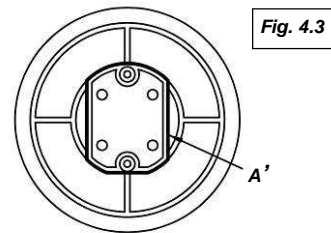b) *Cutting outline* : (see fig. 4.3) If necessary (thin transtibial socket, aesthetic matters...), excess material can be cut off following outline **(A')** instead of outline **(A)** described in chapter E.


F.2.2.) LAMINATION (see fig. 4.4)

Unlike in "standard" finishing, lower side **(B)** of part **(1.1)** must necessarily be laminated (fabrics + resin) and the two following conditions must be met :

- a) lower side **(S)** must be perfectly flat to provide a stable support for connector **(2)**
- b) lower side **(S)** must be perfectly parallel to side **(B)** of part **(1.1)** to keep the alignments already performed.

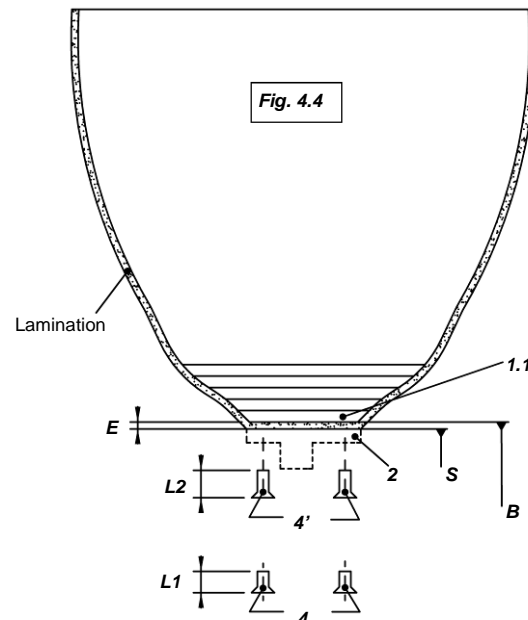CAUTION !!! To guarantee correct screwing and to compensate for lamination thickness **(E)**, delivered screws **(4)** with length **(L1)** must not be used to connect link piece **(2)**. Instead of them you must use other screws **(4')** whose length **L2** is calculated as follows : $L2 = L1 + E$.

6, rue DE LA REDOUTE - Z.I. SAINT-APOLLINAIRE
 B.P. 37833 - 21078 DIJON CEDEX - FRANCE
 PHONE : +33 3 80 78 42 10 - FAX : +33 3 80 78 42 15
 www.proteor.com - htorthopedie@proteor.com

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