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
TWO-PART COSMETIC COVER 1M11294 FOR 1M112 AND 1M113 (Instructions for use)

1) PACK CONTENTS

- a soft polyurethane foam cap (1)
- an intermediate block in hard polyurethane foam (2)
- two screws for plastics (3)
- an assembling metal plate (4)


2) ALIGNMENT

- a) Align the socket and the knee as indicated in the knee instructions sheet 1M11299-12/06.
- b) Remove the knee taking care to keep the previously set alignment.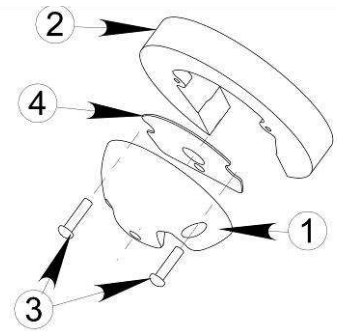


CAUTION : As later angular adjustments are not possible (they may cause a stop between the knee and the cap), the final alignment of the prosthesis must be set during trial fitting with the patient before preparing 1M11294.


3) SHAPING THE INTERMEDIATE BLOCK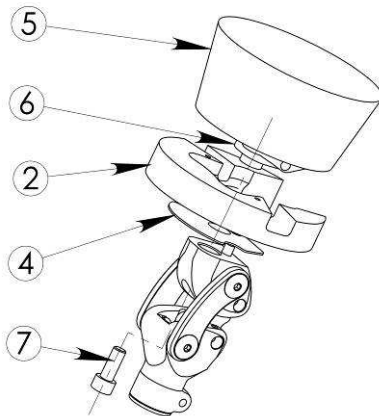

- a) Install the knee on socket 5 equipped with its linking plate 6 by inserting (as indicated on the diagram) metal plate 4 and intermediate block 2, whose upper face has been covered with resin glue ref.PC232-5. Also apply this resin glue to the lower part of socket 5, but only on the surfaces of block 2 and socket 5 that are opposite to each other. Fix the assembly by screwing screw 7. Take care to keep the alignment.

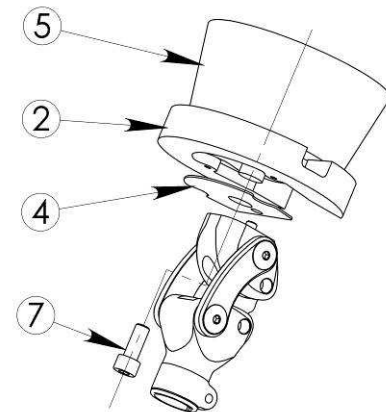
- b) When the glue is dry, unscrew screw 7 and remove the knee. Remove also plate 4 (it will be no longer useful).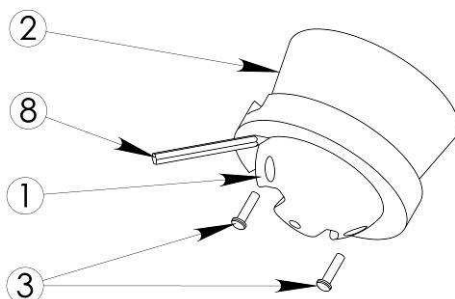


- c) Install cap 1 under intermediate block 2. Screw the 2 screws 3 in the relevant holes of block 2. Using a pencil or a marking tool 8, draw the outline of cap 1 on intermediate block 2.

Lamination thickness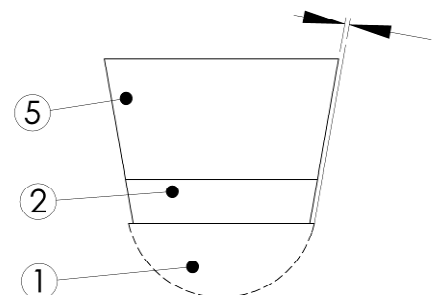

- d) Remove cap 1, then shape intermediate block 2 and socket 5 inside the outline previously drawn. The space left should correspond to the thickness of the lamination performed in point e).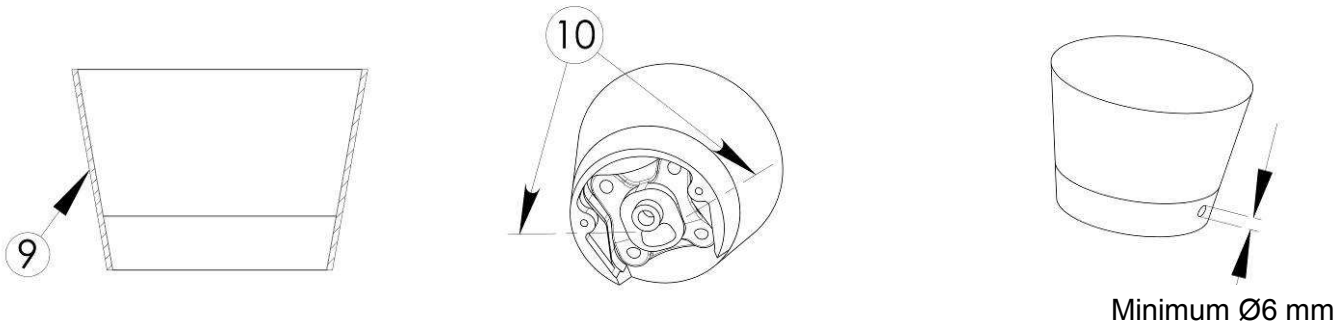

e) Proceed to lamination 9 as usually.

f) With a pencil draw two lines on the lower face of the intermediate block to mark location of the two screws 10 of the linking plate.

g) On the socket sides, 11 mm above the lower end of the intermediate block and according to the axes previously drawn with a pencil on the lower face, drill 2 holes of minimum \varnothing 6 mm. These holes will be used only to assemble/disassemble the knee (please see instruction sheet for 1K03/1K30).



4) ASSEMBLING THE SHIN

Install socket 5 and cap 1 on the knee. Fix the assembly with screws 3 and 7. Place the tube.

IMPORTANT : this cosmetic cover is to be used exclusively with the following shins :

- . shin 1G22 for knee 1M112
- . Shin 1G32 for knee 1M113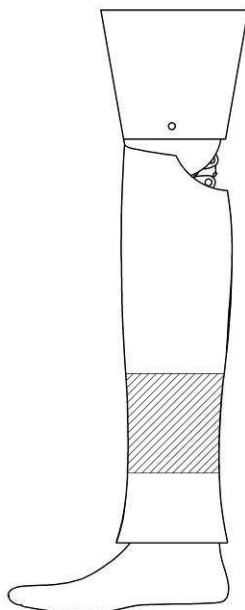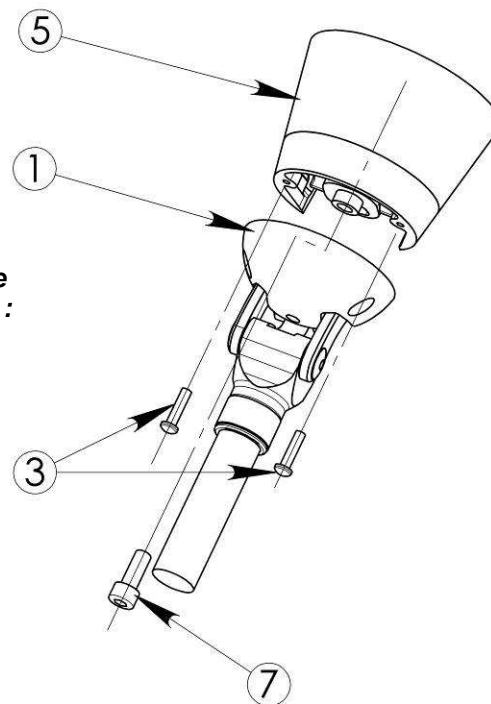

Don the shin on the prosthesis and make it lie flat on the cap. The distance between the upper front border of the shin and the lower border of the socket must not exceed 5 mm. Determine the excess shin length. Remove the shin and cut off a horizontal slice that corresponds to the excess length (we recommend to cut off the slice in the area indicated with streaks on the diagram). Stick the two parts and shape the shin to get an aesthetic connexion area. Re-install the shin on the prosthesis. Then shape its upper part until it matches the cap shape and its lower part until it becomes consistent with the foot shape.

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