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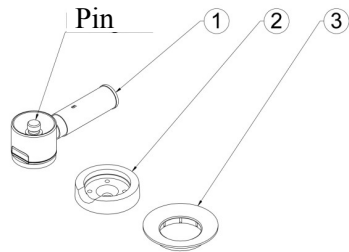
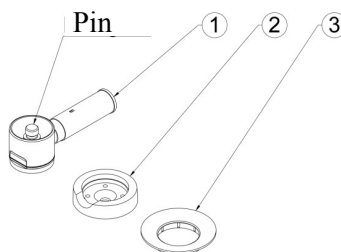
IS117 DISTAL LOCKING KIT INSTRUCTIONS FOR USE

I - KIT CONTENTS :

- Lock (1)
- Linking plate (2)
- Cup (3)
- Screw fittings.

You will need IS112 kit (centring tool), IS119 kit (forming tool) and possibly a socket adapter.

II - ASSEMBLING :

⚠ Do not use another pin than that included in the kit

1) Flatten the end of the plaster positive (fig. 1) and fix the IS112 centring tool (fig. 2) with two small nails (not supplied). If needed, re-shape the contour with plaster or compound. Smooth the assembly.

2) Install the direction rod in the centring tool (fig. 2). Position IS119 tool on the rod (fig. 4) and check its direction according to the place selected for the release button.

3) For a laminated socket : isolate the assembly with a PVA film or cone (PL030-x) with the usual method.

4) If a socket adapter is needed, place it in the required position (fig. 5).

5) Make the socket with the usual materials.

6) Surface the area of the socket where the release button will be located.

7) Remove the positive and take IS118 and IS112 tools.

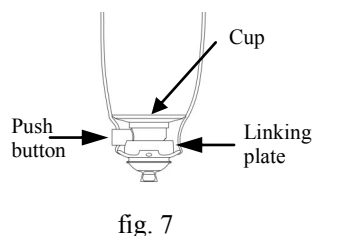
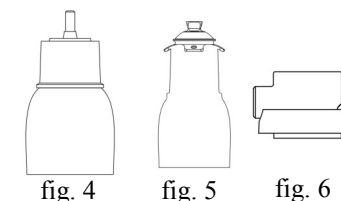
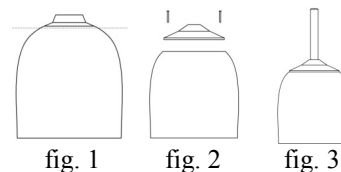
8) With the 4 supplied screws, assemble linking plate (2) and lock (1). (fig. 6).

9) Unscrew the release button and mount the assembly into the socket with the most appropriate method.

10) Finish assembling the lock. Adjust the length of the push-button of the lock according to the cosmesis. Install cup (3) in the bottom of the socket (fig. 7). Adjust and adapt the socket.

III - RECYCLING :

The parts of the kit are made of stainless steel and plastics. These parts must be recycled in compliance with the laws in force.

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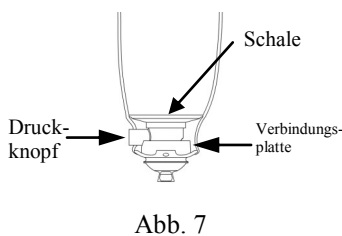
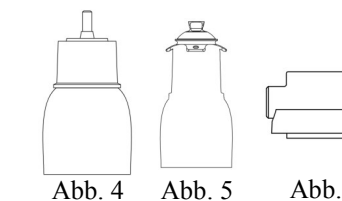
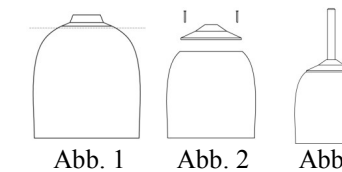
⚠ SAFETY INSTRUCTIONS FOR THE PATIENT

- Do not let foreign bodies inside the socket or the locking mechanism.
- Clean regularly by blowing or sucking.
- If the parts have been dipped or sprinkled, empty the remaining liquid out and let dry.
- In case of metallic component corrosion or doubts about the behaviour of the prosthesis, have it controlled.
- Never use grease, oil or any other lubricant.
- Do not use another pin than that included in the PROTEOR kit (be very careful when changing the liner for example).

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