#### 4-Alineamiento: dib. 3

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## Dyna J Foot 1A410 Assembling Instructions

#### SAFETY INSTRUCTIONS FOR THE PATIENT

- The cosmetic cover is guaranteed for 6 months. It must be replaced if it tears or if there is clearance between the structure and the cosmetic cover.
- Do not let foreign bodies or dust inside the foot.
- If the foot has been dipped or sprinkled, empty the remaining liquid out and dry the foot rapidly. In case of metallic component corrosion, have your prosthesis controlled.
- Use temperature :  $-20^\circ / +60^\circ\text{C}$
- An inappropriate use of the foot (top level sport or frequent overload) can damage components and be dangerous for you. If you discover an abnormal behaviour of your foot or changes in your foot features, contact a prosthetist.

#### 1- Description and functions

Dyna J foot includes three main components : two composite parts and a cosmetic cover. The J-form strut and the sole allows energy storage and return, and provides flexibility in inversion-eversion to ensure adaptation to the ground. The polyurethane cosmetic cover upholsters the structure and ensures a good position in the shoe. Thanks to these components the foot behaviour is flexible and dynamic.

#### 2- Specifications

Dyna J foot is designed exclusively for lower limb prostheses. It is appropriate for active patients up to 100 Kg or very active patients up to 90 Kg. The heel height can be adjusted from 5 to 22 mm. Dyna J includes a tube and an intermediate piece  $\varnothing 34$  that will be stuck to allow the use of standard connectors  $\varnothing 34$ .


#### Contents

- 1 J-form strut
- 2 half-soles
- 2 screws, 2 washers  $\varnothing 6$
- 1 sticking kit
- 1 intermediate piece  $\varnothing 30/\varnothing 34$
- 1 sock
- 1 cosmetic cover
- 2 reinforcing plates
- 1 malleolar clip.

#### 3- Assembling : (without the patient) fig. 1 and 2

Place assembling tee 1A41004 in the vice (or a  $250 \times 30 \times 4$  plastic plate) and put the half-soles on each side of the plate. Use wedges (Heel height  $H + 2$  mm) to simulate heel height. Position the J-form strut, the reinforcing plates, the washers (with spheric side on the bevel) and the screws according to fig.2. Tighten the screws with a 10 mm hex wrench to get a  $8.5 \pm 0.5$  Nm tightening torque and check dimension A.


Fig. 1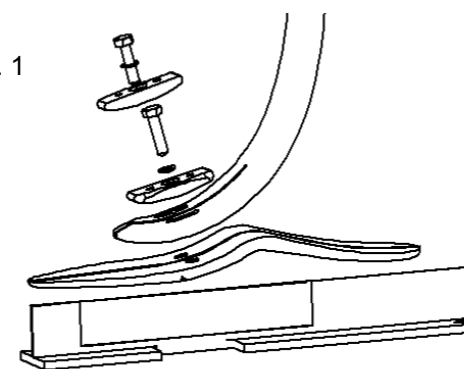

#### Module selection guide

| Activity | Patient weight (Kg) |          |          |          |           |
|----------|---------------------|----------|----------|----------|-----------|
|          | 54 to 62            | 62 to 70 | 70 to 80 | 80 to 90 | 90 to 100 |
| Moderate | 1                   | 1        | 2        | 3        | 4         |
| Normal   | 1                   | 2        | 3        | 4        | 5         |
| High     | 2                   | 3        | 4        | 5        | N/A       |

Fig. 2

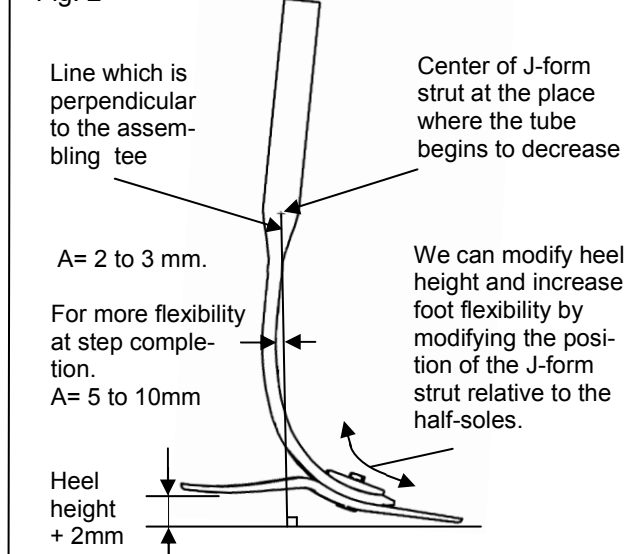6, rue de la Redoute - Z.I. SAINT-APOLLINAIRE  
B.P. 37833 - 21078 DIJON cedex - FRANCE  
Tél. 33 (0) 3 80 78 42 10 - Fax 33 (0) 3 80 78 42 15  
E.mail : ht.orthopedie@proteor.com - Internet : www.proteor.com

**4-Alignment : fig. 3**

To align the prosthesis you must be able to modify the foot antero-posterior position and tilting ( $\pm 9^\circ$ ).

Slip the cosmetic cover on the foot, then put them into the shoe. Cut the tube at the desired length + 1 cm. Sand the tube end until you reach the fiber, then remove dust and degrease. Apply on the tube and on the intermediate piece  $\varnothing 30/\varnothing 34$  some of the glue supplied, fit them and let dry. Place a  $\varnothing 34$  tubular adaptor with a 35 mm minimal depth.

You can also use 1D47 tube holder for sticking. In that case, use 1D41 tube holder at fitting trial, then at finishing stick 1D47 connector.

In order to avoid foot rotation, you must necessarily use a connector for sticking or the supplied intermediate piece.

Finish the assembly so that the position indicated in fig 2 is kept. Place the foot forward or backward, so that the load line falls at the posterior third of the foot.

If you cannot find easily your patient load line, you can consider that it falls from 2 to 3 cm in front of the axis of the internal malleolus of the sound ankle.

**5-Dynamic adjustment**

Walking must be the least tiring as possible for the patient. During this fitting trial, cut the tube and the intermediate piece to adjust the prosthesis height and check that the intermediate piece protrudes from the tube-holder.

| Problems  | Solutions   |
|---|---|
| Knee hyper-extension. The patient feels as if he raises on his forefoot.  | Make a posterior translation of the foot relative to the socket.                                  |
| The prosthesis can hardly be controlled, the foot returns few energy at step completion.  | Make an anterior translation of the foot relative to the socket.                                  |
| Knee hyper-extension in static vertical position.   | Only in that case we can adjust equinus adjustment by adding a few mm to the heel height (fig 1). |
| <b>Note :</b><br>- Increasing equinus provides a more dynamic alignment. It increases energy return at step completion, but decreases stability when the patient does not walk. |   |

**!** When the glue is dry, put the supplied protecting sock on the struts and slip the cosmetic cover on them. Check that the soles are well clipped inside the cosmetic cover. To avoid rubbing noises, pull the sock onto the strut and fix it with a fine string for example.

**Installation of the malleolar clip :** Cut the cosmetic foam to the desired length, position it on the foot and draw the malleolus contour. Inside the foam mill a space that will receive the selected connector and the foot part protruding from the cover, and at the same time take care that thickness be sufficient to allow for sticking. Position the device carefully and clip it into the cosmesis. Clean contact surfaces with isopropyl alcohol and stick the device on the foam with XC090 neoprene adhesive. After drying, finish the outer shape as usually.

**7-Tools/accessories**


Assembling tee 1A41004 allows for an easier assembly of the foot.

Shoe horn 1A40019 allows to remove and re-install the cosmetic cover.

Spare cosmetic cover 1A40020-D22 including the sock and the malleolar clip. Side  $\rightarrow$  Size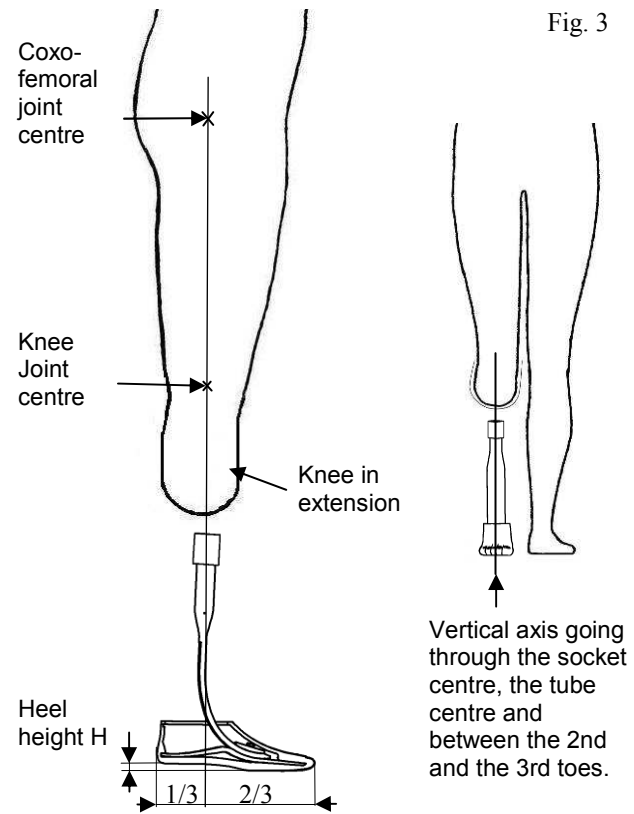

**6-Finishing : Sticking must be carefully performed : the mechanical properties of the foot depends on it**

After adjustment, remove the cosmetic cover and mark the position of the half-soles relative to the J-form strut. Loosen the screws with a 10 mm hex wrench. Sand the surfaces that will be stuck until you reach the fiber, remove dust and degrease. Position the half-soles on each side of the assembling Tee. Apply glue to the parts and the reinforcing plates and take care to fill up completely the apertures of the J-form strut. Assemble the screws and the washers (spheric part on the bevel of the reinforcing piece) and tighten to get a  $8.5 \pm 0.5$  Nm tightening torque. Take care to follow the instructions for use of the supplied glue (setting time, etc...) Remove the foot from the assembling tee and clean glue stains. There must be no glue in the slit.

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