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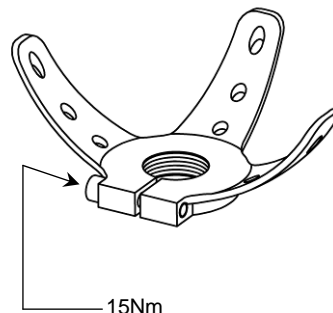
1K17999-0411

I. USE

- This component is a steel lamination anchor with a M36x1.5 female thread that will be placed on components with a M36x1.5 male thread, and integrated into the laminate of a socket.
- 1K179 anchor is convenient for patients up to 125 kg, carried load included.

II. ADAPTATION TO THE SOCKET FORM

- The prongs of 1K179 anchor can be shaped with your usual method to match the socket form.
- **Caution** : when shaping the anchor prongs, take care not to bend them with too low curvature radius and do not use forming tongs whose claws have sharp edges. To prevent thread distortion during prong shaping, we recommend using 1K17910 tool. If these recommendations are not observed, the anchor may be damaged and crackings in the steel may occur during use.

III. SOCKET LAMINATION

- Protect the anchor thread with plastiline or with 1K17910 tool (please order separately).
- Before placing the anchor on the plaster model, cover the latter with several layers of glass or carbon fabrics.
- Position the anchor and check its alignment very carefully, then put some layers of glass or carbon fabrics over it. Tie off these fabrics layers at the thread base.
- Perform lamination with your usual method.
- After resin polymerization, remove material excess to free the access to the anchor thread.

Caution! In case of insufficient laminate reinforcement, anchor 1K179 could break out from the socket.

IV. CHECK

Functions of 1K179 anchor are to be checked in regular intervals and at least once a year.

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